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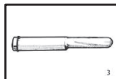
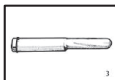
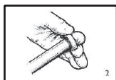
LATEX PROBE COVERS

INSTRUCTION FOR USE

Non-sterile Natural Rubber Latex Protective Transducer Cover.

WARNING Transducer cover is a medical device used by healthcare provider trained in the use of ultrasound. It is intended for barrier protection during invasive medical examination and diagnosis using transducer probe. Before use, you should be trained in ultrasonography. **DISPOSE** of single use components as infectious waste. **DO NOT REUSE** transducer covers. Reuse or reprocessing may create a risk of contamination of the device and cause patient infection or cross-infection. **USE ONLY WATER BASED LUBRICANT**, avoid petroleum or mineral oil based lubricant that may harm the covers. Disinfect the transducer probe after each examination could reduce the probability of cross-contamination. Report any serious incident occurred from using the device to the manufacturer and the relevant national competent authority. Prevent cross-contamination of probes and consequently reduce the risk of cross infection in patients undergo invasive ultrasound examinations and diagnoses (i.e transvaginal and transrectal scan). **This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.**

INSTRUCTION

1. Carefully tear off the foil pack. Make sure not to damage the cover.
2. During removal from foil, avoid damage to the cover by sharp edges, fingernails and etc.
3. Place the cover over the transducer. Pull cover tightly over the transducer to remove wrinkles and air bubbles, taking care to avoid puncturing the cover.
4. Inspect the cover to ensure no holes or tears.

	Store in a cool dry place		Keep away from sunlight		Use by date
	Manufacture date		Manufacturer		Consult Instruction for use
	Do not use if package is damaged		Do not reuse		Batch Code
	European authorized representative		Medical Device		

Date of Revision : 15th March 2023

Karex Industries Sdn. Bhd.
 PTD 7906 & 7907,
 Taman Pontian Jaya,
 Bt.34 Jalan Johor,
 82000 Pontian, Johor,
 Malaysia.

VULKAN – Medical, a.s.
 U gumovky 439
 463 34 Hradek nad Nisou
 Czech Republic
 DIC / VAT:CZ27226158
www.vulkanmedical.cz

EC REP

Advena Ltd
 Tower Business Centre,
 2nd Floor, Tower Street,
 Swatara, BKR 4013, Malta